AQUOLAB®

AQUOLAB® PROFESSIONAL

- IT** Manuale di istruzioni
- EN** User's manual
- ES** Manual de instrucciones
- P** Manual de instruções

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
AQUOLAB® PROFESSIONAL - DESCRIPTION OF COMPONENTS

- a) nozzles (10 sterilised disposable nozzles)
- b) command management LCD touch screen
- c) push-button to turn the water flow ON/OFF
- d) device handle
- e) tank (300ml)
- f) tank cover

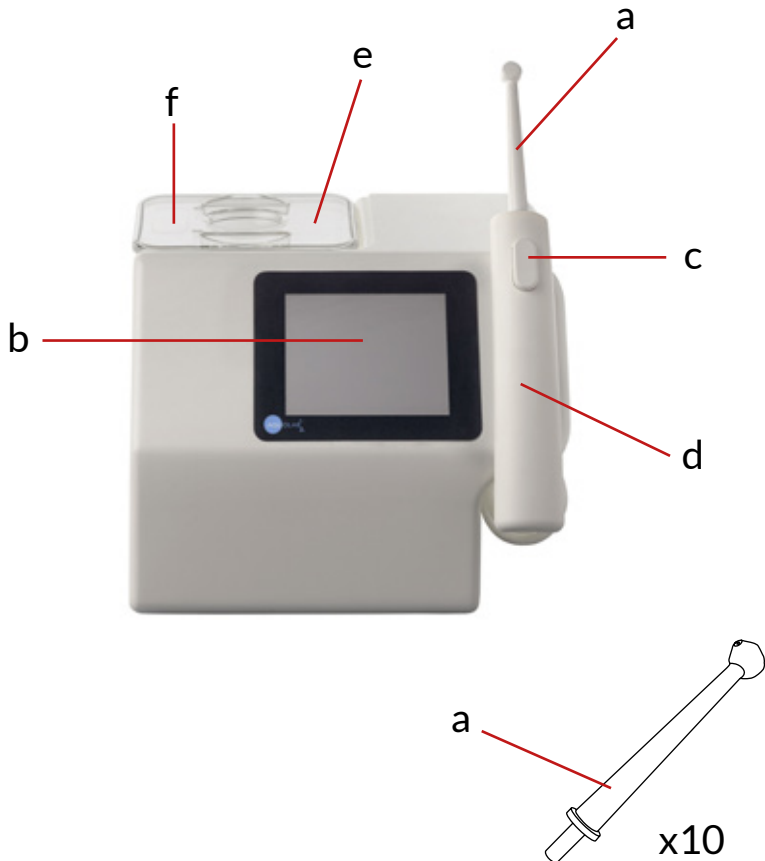Thanks to its antibacterial, antimicrobial, immunomodulating and anti-inflammatory properties, ozone is able to rapidly decompose bacteria and viruses, preventing inflammatory processes and promoting healing at the cellular level.

Aquolab's jet of water and ozone eliminates bacteria and viruses from the patient's mouth gently, painlessly and in just a few seconds, without leaving any chemical residue, because the ozone transforms into oxygen after a few seconds. Aquolab produces ozone internally, in predetermined amounts, in full compliance with European safety standards (TLV STEL and TLV TWA), with no need for external oxygen tanks. The ozone mixed with water is particularly effective and, moreover, it is not released into the air, and therefore cannot be inhaled by the patient, a fact which makes this device even safer. Aquolab can operate effectively using regular potable water, but it can also be used with distilled water or with a physiological solution.

The protocols for use list the water and ozone levels, for easy, safe and fast treatments of a wide range of case types.

FIELDS OF APPLICATION:

- Non-surgical periodontal therapy
- Support (disinfection pre- and post-surgery) for oral surgery, mucogingival surgery, implant surgery
- Support (disinfection) for root canal therapy
- Professional oral hygiene (pre- and post-treatment)
- Oral hygiene treatments for patients with implants and prostheses
- Oral hygiene treatments for patients undergoing orthodontic treatments
- Peri-implantitis treatment
- Treatment of candidiasis and stomatitis

Any other use than the intended one constitutes an improper use. Therefore, the manufacturer cannot be held liable for any damage caused by an improper, incorrect and/or unreasonable use or if the device is connected to electrical installations which do not comply with safety regulations in force.

The Aquolab dental water jet is not suitable for use in pure oxygen-rich spaces or where there are flammable anaesthetic mixtures.

GENERAL RULES FOR THE CORRECT USE OF THE AQUOLAB DEVICE

The protocols for using Aquolab are subject to continuous updating and revision.

Visit www.aquolab.com/professionisti to download the latest version or request a copy at info@aquolab.com

ACTIVATION

You must read the instruction manual and safety rules carefully before use.

- Place Aquolab on a horizontal surface.
- Connect the handle (tube) to the Device.
- Connect the power supply charger to an easily accessible outlet.
- Connect to an electrical system constructed in accordance with the safety standards.
- Now your Aquolab Professional is ready.

ACCESSORIES

As standard equipment you will receive 10 sterilised disposable nozzles: 7 with a 0.6 mm diameter (coloured nozzles) and 3 with a 0.8 mm diameter (white nozzles). We recommend using the protocols as a reference for the correct use of the two types of nozzles. Indicatively, the nozzles with a diameter of 0.8 mm are suitable for patients with heightened dentine sensitivity and for children. The package you will receive also contains a blister package with 3 vials of de-scaling solution for ordinary maintenance (see the section on Maintenance).

Disposable sterilised nozzles with diameters of 0.6 mm and 0.8 mm are available at www.aquolab.com.

The handle is detachable. If it is damaged or requires replacement, you can replace it without having to replace the entire device.

HOW TO USE THE AQUOLAB® DENTAL WATER JET

1. When using for the first time or after a long period of inactivity, perform a vacuum cycle before using your dental water jet
2. The device switches off automatically after 1 min/1.5 min to prevent water-free operation
3. Fill the tank with water, about 300 ml
4. Insert a sterilised disposable nozzle into the handle
5. Use the LCD touch screen to adjust all the settings related to water pressure and ozone quantity. (b.)
6. Press the button activating the water flow. (c)
7. Direct the nozzle as needed and press button D to activate/deactivate the water jet

HOW TO REGULATE WATER AND OZONE LEVELS

To regulate water and ozone levels, use the HOME PAGE to access the PLAY page (see figure) and use the relevant arrows. You can input and save patients/treatments and their respective ozone/water levels:

1. on the "HOME PAGE" (fig. 1) select the "+" button.
2. The "KEYPAD" page (fig. 2) will open, where you can type in the name of a patient or treatment. When you have finished typing, confirm by pressing the "OK" button.
3. The patient/treatment you have input will now appear in the list on the "HOME PAGE". To set the ozone and water levels for that patient/treatment, click on the arrow next to the name to access the PLAY page (fig. 3).
4. This is where you can set the levels. Click OK to save them.

These are just general recommendations. For specific needs, contact our Customer Service by calling number: +39 02 366 86 416 or writing an email to: assistenza@aquolab.com

HOW TO USE THE LCD TOUCH SCREEN

HOME PAGE (FIG. 1)

Maintenance notification. It appears only when maintenance is needed

Go to the "Maintenance" page

Add a user. Request should be confirmed through a pop-up. If the request is confirmed, a system keypad appears

Go to the "Play" page

Scroll back on the users' list (by 3 users)

Scroll forward on the users' list (by 3 users)

Go to the "Set Up" page

Scroll back on the users' list by one letter

Scroll forward on the users' list by one letter

KEYPAD PAGE (FIG. 2)

Go back to the previous page

Qwerty Keypad

Confirm and go to next page

PLAY PAGE (FIG. 3)

Modify the level of ozone produced (0-3)

Modify the water level (1-2)

Go back to the "Home Page".

Delete the current user. Request should be confirmed through a pop-up

Modify current user's name through the system keypad

Save the water and ozone levels for the current user. This button only appears when levels are modified

Start water and ozone flow

Flow timer based on the set water level

09:00 Alighieri Dante 03 Apr

02 Water 01 Ozone 01:40

Home X Edit OK

PLAY PAGE - FLOW IS ON (FIG. 4)

Go back to the "Home Page". The flow stops.

Stop the water and ozone flow

Flow timer based on the set water level. Time decreases while the flow is on.

Delete the current user. Request should be confirmed through a pop-up. If the request is confirmed, the flow stops.

Modify current user's name through the system keypad. The flow stops.

09:00 Alighieri Dante 03 Apr

02 Water 01 Ozone 1:39

Home X Edit OK

PLAY PAGE - FAULT STATUS (FIG. 5)

Go back to the "Home Page"

No actions allowed


Alerts you to the identified problem

09:00 Alighieri Dante 03 Apr


02 Water 01 Ozone Ozone

Home X Edit OK


SET-UP PAGE (FIG. 6)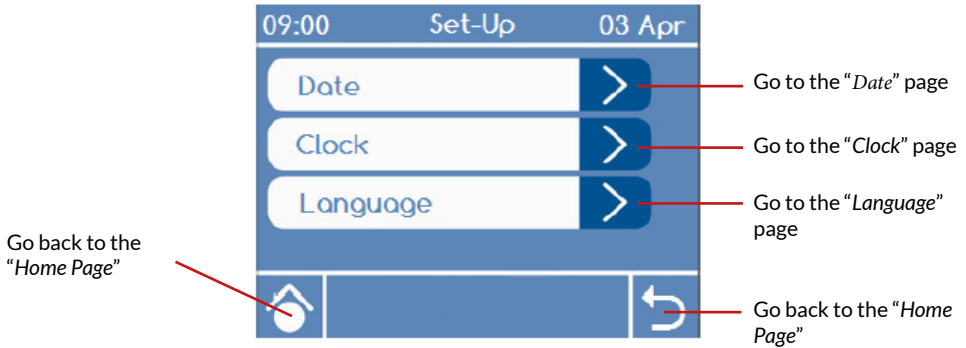


DATE PAGE (FIG. 7)

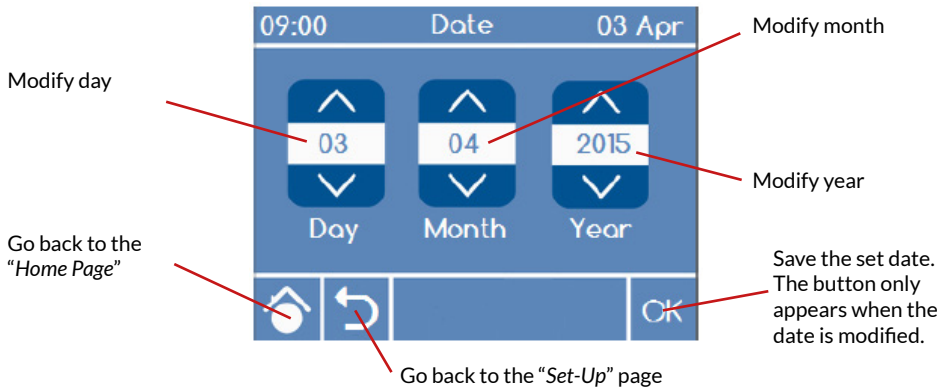CLOCK PAGE (FIG. 8)

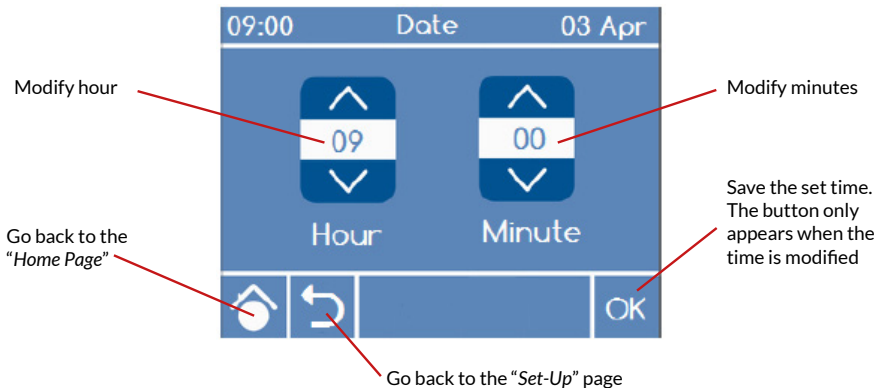LANGUAGE PAGE (FIG. 9)

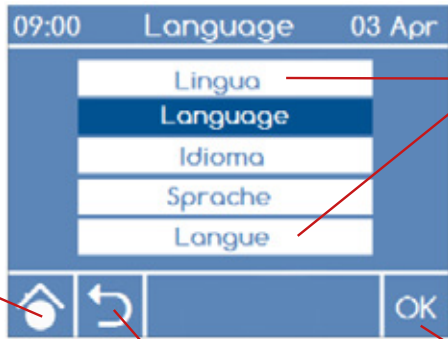
Go back to the "Home Page"

Go back to the "Set-Up" page

You can choose from among the set languages

Save the set language. The button only appears when the language is modified

MAINTENANCE PAGE (FIG. 10)

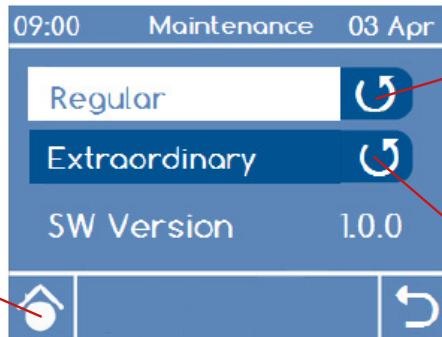Go back to the "Home Page"

Reset an ordinary maintenance request

Reset: extraordinary maintenance request. A keypad will appear to insert a password. To be used by maintenance staff only

IMPORTANT

Periodically check the condition of the external and visible parts of Aquolab. If you notice that the housing, the charger or the cable is damaged, do not use the device. Contact customer service (+39 02 366 86 416 – assistenza@aquolab.com).

Do not use the charger or cables if they are damaged.

Replacements are available for all of the external parts. Aquolab will automatically warn you of any malfunctions that occur in its internal components. If the solution to the problem cannot be found in this manual, contact customer service.

IMPORTANT WARNINGS

- Use only with the provided EM1024RE-24 power supply charger.
- Fill the tank with water. DO NOT use the device without water or if it fails to dispense water.
- Use fresh water (not stagnant) every time.
- Use water at room temperature (avoid warm or cooled water)
- Use a new sterilised disposable nozzle for each patient
- Its use is intended for the oral cavity only.
- DO NOT use the device if its external parts or the charger are damaged.
- Use only for the intended use.
- Do not modify the device in any way
- De-scale the device regularly, using only the specific product indicated.

DIRECTIONS FOR ELECTROMAGNETIC COMPATIBILITY

Since Aquolab is an electro-medical product, you need pay attention to the following tables, in order to let the device operate safely in accordance with the information about electro-medical machines.

It is important to remember that portable and mobile radio communication can affect the device's operation. It is recommended that any electro-medical device – therefore, including Aquolab – not be used near or stacked with other equipment.

TABLE 1 - GUIDELINES AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC EMISSIONS

RF emissions CISPR 11	Group 1	The device "AQUOLAB PROFESSIONAL" uses RF energy only for its own internal operation. Therefore, its RF emissions are very low and are not likely to cause any interference to nearby electronic equipment.
RF emissions CISPR 11	Class B	The device "AQUOLAB PROFESSIONAL" is suitable for use in all environments, including home environments and those directly connected to the public low-voltage power supply network that also supplies buildings used for residential purposes.
Emission of harmonics IEC 61000-3-2	Class A	
Emission of voltage fluctuations and flicker IEC 61000-3-3	Compliance	

TABLE 2 - GUIDELINES AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC INTERFERENCE RESISTANCE

Electrostatic Discharge (ESD) IEC 61000-4-2	± 6kV contact ± 8 kV air	(IMMUNITY) ± 6kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Fast transient/electrical bursts IEC 61000-4-4	± 2kV for power supply lines	± 2kV for power supply lines	The mains power supply quality should be that of a typical commercial or hospital environment.
Surges IEC 61000-4-5	± 1 kV differential mode	± 1 kV differential mode	The mains power supply quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations of the power supply lines IEC 61000-4-11	< 5% U_i (> 95% dip in U_i) for ½ period 40% U_i (60% dip in U_i) for 5 periods 70% U_i (30% dip in U_i) for 25 periods < 5% U_i (> 95% dip in U_i) for 5s	< 5% U_i (> 95% dip in U_i) for ½ period 40% U_i (60% dip in U_i) for 5 periods 70% U_i (30% dip in U_i) for 25 periods < 5% U_i (> 95% dip in U_i) for 5s	The mains power supply quality should be that of a typical commercial or hospital environment. If the user of the device "AQUOLAB PROFESSIONAL" requires continuous operation even in the case of power supply interruptions, it is recommended to connect the device "AQUOLAB PROFESSIONAL" to an uninterruptible power supply or a battery.
Power frequency (50 Hz/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	-

TABLE 4 - RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF TELECOMMUNICATIONS DEVICES AND THE "AQUOLAB PROFESSIONAL"

Maximum output power rating of transmitter	Separation distance according to frequency of transmitter m		
	From 150 kHz to 80 MHz $d = 1.2 P$	From 80 MHz to 800 MHz $d = 1.2 P$	From 800 MHz to 2.5 GHz $d = 2.3 P$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

TABLE 3 - GUIDELINES AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC INTERFERENCE RESISTANCE

Interference Resistance (Immunity) Test	IEC 60601 EN 60601-1-2	Compliance Level	Electromagnetic Environment - Guidelines (IMMUNITY)
Conducted RF EN 61000-4-6	3 V eff From 150 kHz to 80 MHz	3 V eff From 150 kHz to 80 MHz	Portable and mobile RF communications devices should be used no closer to "AQUOLAB PROFESSIONAL" or any of its components (including cables) than the recommended separation distance determined using the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2\sqrt{P}$ 150 kHz to 80 MHz Recommended separation distance: $d = 1.2\sqrt{P}$ 80 MHz to 800 MHz Recommended separation distance: $d = 2.3\sqrt{P}$ 800 MHz to 2.5 GHz
Radiated RF EN 61000-4-3	3 V/m From 80 MHz to 2.5 GHz	3 V/m From 80 MHz to 2.5 GHz	Where P is the maximum output power rating of the transmitter in watts (W) according to the information provided by the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, must be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 

AQUOLAB PROFESSIONAL® MAINTENANCE

Disconnect the device from the supply mains before any maintenance activity

ORDINARY CLEANING

Dry the water tank (f.) with a cloth after every use to prevent limestone scale build-up.

ORDINARY MAINTENANCE - ANTI-SCALING TREATMENT

Aquolab Professional alerts you to the need for ordinary maintenance using the symbol **(i)** (see the "Home Page" figure). By clicking it, you will open the "Maintenance" page.

Remember that, at least once a month or when Aquolab gives you this signal, you need to run a de-scaling cycle with the specific de-scaling solution supplied in vials (Aquolab De-scaling solution) which you can purchase directly from the www.aquolab.com website. The de-scaling cycle allows you to remove scale build-up resulting from calcium and magnesium salts in water. This will maintain continuing device efficiency. To avoid damaging the device, do not use products other than the Aquolab de-scaling kit.

Follow these steps to correctly use the Aquolab De-scaling solution:

- Shake the vial before using it.
- Pour the contents into the dental water jet tank and fill it with water.
- Switch the device on and let it dispense water for approximately 15 sec.
- Wait 5 minutes and repeat this operation for a further 15 sec.
- Repeat the operation until the tank is completely empty.
- Rinse the tank by filling it with water only and letting it flow out at regular intervals until the tank is completely empty.

To reset the "Ordinary maintenance" alarm and make Aquolab Professional ready for use again, simply press the reset button on the "Maintenance" page.

HOW TO REPLACE A DAMAGED HANDLE

Carefully detach the cable on the side of the device so as not to damage the outlet fitting. Replace the handle with a new one, carefully inserting the cable into the outlet fitting.

EXTRAORDINARY MAINTENANCE - REPLACEMENT OF THE OZONE PRODUCER

The production of ozone at the required levels is guaranteed until the **(i)** warning light comes on (see the "HOME PAGE" fig.), which will cause the "MAINTENANCE" page to open. Alternatively, a malfunction or breakdown of the ozone producer may be indicated as shown in the figure "PLAY - FAULT STATUS". Production of ozone at the required levels lasts for approximately 3000 cycles of use. The device must be sent back to Customer Service for replacement of the Ozone Producer. It is not possible to independently reset the "Extraordinary Maintenance" alarm. However, even if this alarm is activated, the device will work continue to work. Nonetheless, the ozone producer must be replaced as soon as possible in order for Aquolab to remain efficient and performing at its best.

IMPORTANT

- Always remember to empty the hydraulic circuit before any transport or long-term storage of the device.
- If the device is going to be returned for extraordinary maintenance, remove the nozzles before returning.
- When requesting a replacement handle, be very careful not to damage the outlet fitting when removing the old handle.
- **These are fundamental requirements. Non-compliant returns may be refused.**

DISPOSAL

Pay particular attention to the regulations on waste sorting and disposal currently in force. The device must not be thrown away as household waste, but has to be handed over at authorized municipal landfills.

The external and replaceable parts of the water jet, such as nozzles and handle, can be recycled normally.

TECHNICAL SPECIFICATIONS

Manufacturer	EB2C srl - Via Savona, 94 - 20144 Milan - Italy
Model	Aquolab Professional
Voltage supply	100-240 V 50/60 Hz 36 W
Class	II
Applied part	Handle (type B)
Weight	0.99 Kg
Dimensions	190 mm x 140 mm x h150 mm
Usage temperature	+5°C ÷ +40°C
Intermittent operation	3 min ON / 1 min OFF
Transport temperature	-25°C ÷ +50°C

EXPLANATION OF LABELLING SYMBOLS

	Type B applied part
	It is obligatory to read this user's manual
	CE marking
	Selective waste sorting is obligatory
	Do not reuse (disposable device - just for some handles)
	Protected against access with a finger to dangerous voltage parts Protected against the entry of vertical water drops

WARRANTY CONDITIONS

- The device is covered by warranty, valid 2 years from the date of purchase, for any faults in workmanship or materials used.
- Under this warranty, Aquolab will replace and/or repair originally defective components free of charge.
- The warranty does not cover the accessories supplied with the device or parts subjected to normal wear.
- The device must be repaired at authorized technical service centres only.
- The device must be sent to the authorized service centre within 8 days after the fault is noticed.
- The user is responsible for shipping costs.
- The user shall bear the expense for any repairs not covered by the warranty conditions.
- The warranty will be rendered void if the device is misused or abused, if the required regular maintenance (anti-scaling treatment) has not been performed, or if the damage cannot be attributed to the manufacturer.
- The warranty does not imply any compensation for any kind of damages to people or property, either direct or indirect, that occurs when the device is not operating correctly.
- The warranty is valid as from the date of purchase, certified by the cash register receipt or the invoice.
- The lack of the properly filled in warranty coupon accompanied by the above receipt/invoice voids the warranty.

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This medical device complies with the provisions of directive 93/42/EEC and its subsequent amendments.

Italian Ministry of Health Registration No. 1573613

Aquolab® is a registered trademark
No. 1231275 – EU 13245535

Industrial Patent No. 0001420117

Duplication is prohibited. Partial reproduction, in any form, is also prohibited unless previously authorized in writing by the Manufacturer.

Technical specifications, description, standard accessories supplied and product appearance may change without prior notice.

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