# Resorbfast

**Sutura chirurgica all'acido glicolico intrecciata e rivestita (a rapido assorbimento)**  
**Sutura quirúrgica a base de ácido glicólico, trenzada y revestida (absorbible)**

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### Descrizione

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**sweden & martina**

Sweden&Martina S.p.A.  
 Via Veneto 10 - 35020 Due Carrare, (PD) Italy  
 Tel. +39.049.91.24.300 - Fax +39.049.91.24.290  
 www.sweden-martina.com

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### ISTRUCTION FOR USE

#### Description

**RESORB FAST** suture is a sterile absorbable medical surgical device of synthetic origin, consisting of a polyglycolic acid multifilament. The braid of the purple filaments (C.I. 60725) and is coated with polycaprolactone and calcium stearate. The **RESORB FAST** suture can accidentally cause an initial mild inflammatory reaction in the tissues resulting in the formation of fibrous connective tissue. The progressive loss of the tensile strength of the suture occurs as a result of a hydrolysis process that degrades the polyglycolic acid, allowing it to be absorbed and subsequently metabolized by the body. Studies performed on animals show that the absorption of the **RESORB FAST** suture is practically complete around the 42<sup>nd</sup> day after implantation (short term absorption means absorption after 40-50 days)

Post implantation	Tensile strength residual
7 days	50%
14 days	0%

**RESORB FAST** suture is available in a range of gauge sizes (USP/EP) and lengths, attached to stainless steel needles of different size and shapes. Further details are described in the general catalog.

**RESORB FAST** suture meets the requirements of the Pharmacopoeia European Union for Braided Sterile Absorbable Synthetic Sutures. Indications **RESORB FAST** suture is indicated where only short-term support is required wound termination with rapid absorption of the surgical suture.

#### Contraindications

Due to the rapid loss of its tensile strength **RESORB FAST** should not be used when long-term wound support is required.

#### Precautions

**RESORB FAST** sutures are to be used in hospitals operating theatres and ambulatories or in any hospital environment by qualified personnel, experienced in technical and surgical procedures where the use of an absorbable suture is required. It is therefore recommended that the surgeon will consider the possible use of non absorbable sutures for the closure of tissues which may undergo distension or stretching or requiring additional support. This suture kind may be inappropriate in elderly, malnourished or weakened patients or in patients suffering from conditions which may delay wound healing process.

#### Recommendations

Care should be taken when handling the suture material to avoid any bending or flattening to the thread due to the application of surgical instruments such as forceps or needleholders. Greater knot security requires standard surgical technique of flat and square knots, to be applied with additional throws as may be indicated by the surgical circumstances and the surgeon's experience. Surgical sutures are disposable medical device and must be disposed of as hospital waste material.

#### Adverse reaction

**RESORB FAST** suture may cause local inflammatory reaction associated to erythema formation.

#### Content

36 sterile envelopes.

#### Warnings

Keep in cool dry place, away from any direct heat.

Expiration date is shown on each single envelope and on the outer box. Disposable.

Do not use if package is damaged.

#### Sterility

**RESORB FAST** sutures are sterilized by Gamma Ray.

Sterile unless package is opened or damaged.

Do not resterilize.

Reserved for the exclusive use by surgical medical personnel, in surgery or in hospital setting.  
Keep out of reach of children.

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Sweden & Martina S.p.A.